

Vena Vario Levant Phlebotomy Chair User Guide

Armrests:

The angle, height and distance of the armrest from the patient can all be adjusted with hand operated adjustable clamping handles (highlighted in green.)

Get-up brackets:

These are to aid larger/older patients to get in and out of the chair and prevents the use of the armrests as an alternative, reducing the chance of damage.

Central footbrake:

The chair can be centrally locked by pressing down on the red side of the footbrake on either of the front two castors. Press down on the green side to unlock.



Height adjustment:

Height adjustment is operated by a hydraulic pump at the rear of the chair.

Press the pump pedal lightly to lift the chair.



Press the pump pedal fully to lower the chair.



Trendelenburg position:

The chair can be tilted to an angle of -11° as in the below image.



This can be operated two ways:



By foot pedal

or



By handles under the seat (highlighted in yellow)